mentoplate[®]

Leading in Orthodontic Implants

psm-medical.com

Quality is never an accident.

It is always the result of high intention, sincere effort, intelligent direction and skillful execution; it represents the wise choice of many alternatives.

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Introduction | Einführung

In collaboration with leading Orthodontic specialists, **PSM** has developed the unique **mentoplate®** System for skeletal anchorage to facilitate maxillary advancement in early Class III treatment. The **mentoplate®** System offers a large variety of high precision titanium plates and screws for bone anchorage.

Especially in early Class III treatments in combination with RPE (**BENEFIT® Hybrid Hyrax®**) the **mentoplates®** offer the possibility to treat children from the age of 8 years. Due to the fact that the fixation is performed in the symphysis mandibulae, there is no need to wait for the eruption of the lower canines.

The plate insertion is made by means of a safe and stable fixation with selfdrilling miniscrews directly in the bone following a small incision. The maxillary protraction starts after finishing the RPE with intermaxillary elastic anchorage. The non-compliance treatment method without the need of a facemask shows excellent clinical results and is based on significant clinical research.

With the introduction of the **mentoplate®** System, **PSM** has considerably enhanced the therapeutic diversity of early Class III treatment.

The ANKA Plates used for maxillary or mandibular anchorage for a large variety of tooth movement applications with the added advantage that many **BENEFIT®** appliances can be attached to the plate giving amazing versatility.

A special highlight are the new **Knechtle-Ludwig plates** for anchorage in mandibular and maxillary regions where a reliable longterm stability is essential for a successful result. 70°-angled screwholes facilitate the oral insertion even in very distal areas.

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The *mentoplate*® System | Das *mentoplate*® System


The *mentoplate*®-System provides a perfect solution for enhanced stability with special orthodontic bone anchored plates for intraoral fixation in the maxilla and the mandible. The System includes Titanium plates which are fixed to the bone surface with selfdrilling screws.

- Intrusion and extrusion of teeth
- Distal/mesial movement
- Space closure
- Maxillary protraction
- Maximum retention force
- Easy fixation with selfdrilling screws
- Reduced risk of tooth damage
- Precise control of tooth movement
- Ability to eliminate facemask
- Recommended for children aged 8 and over

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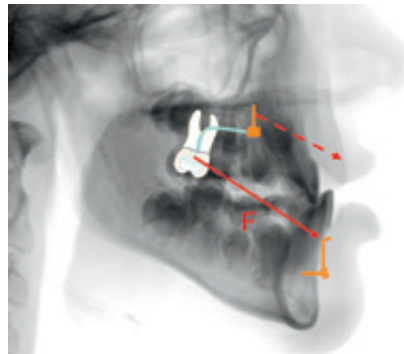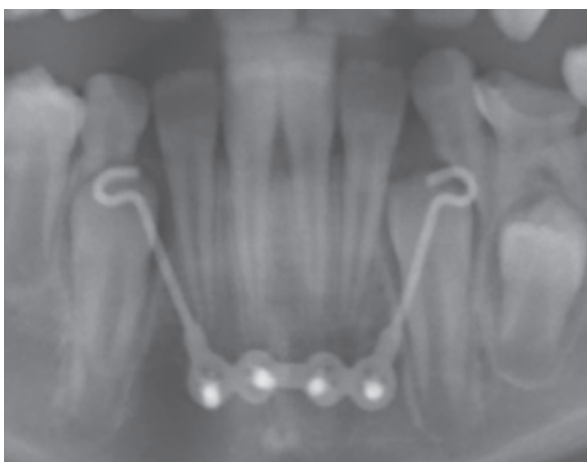
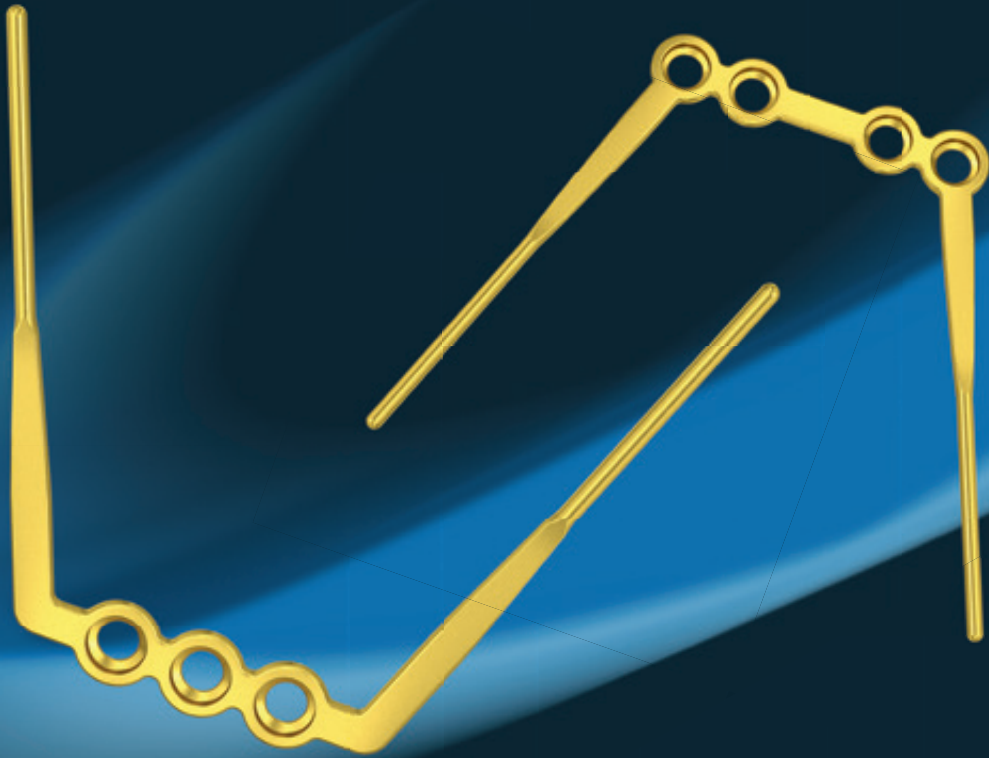
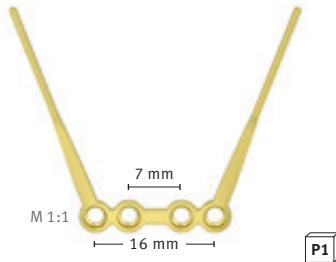
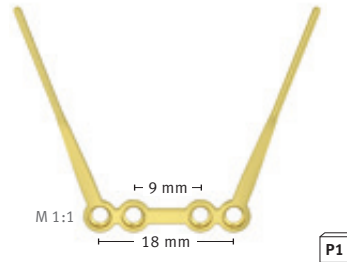
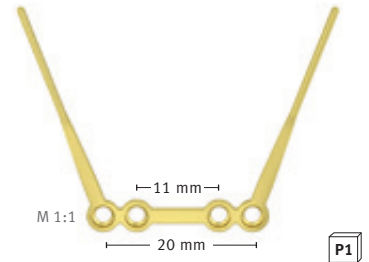
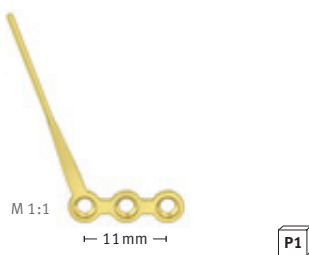
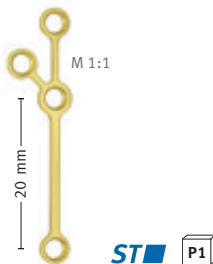
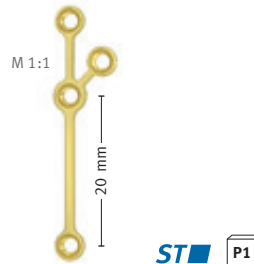
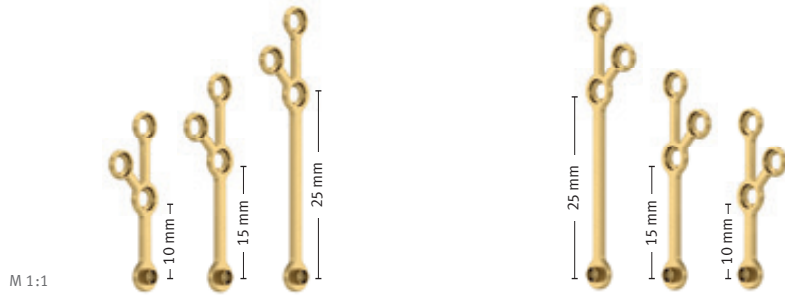
Abutments for Anka Plates | Abutments für Anka Platten

33-54430

BENEFit® Abutment, Standard
BENEFit® Abutment, Standard

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33-54450

BENEFit® Abutment, with 1 bracket
BENEFit® Abutment, mit 1 Bracket

33-54452

BENEFit® Abutment, with 2 brackets
BENEFit® Abutment, mit 2 Brackets

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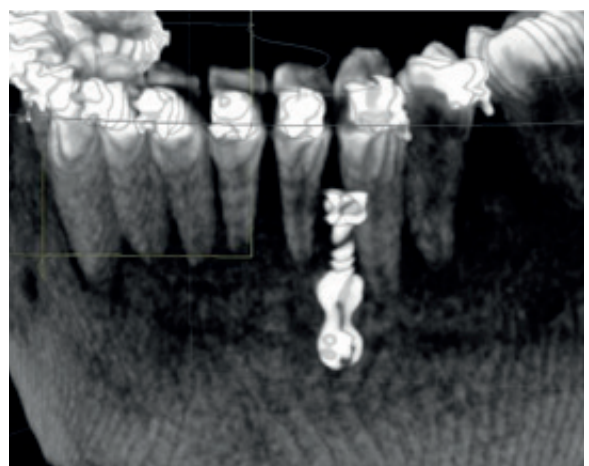
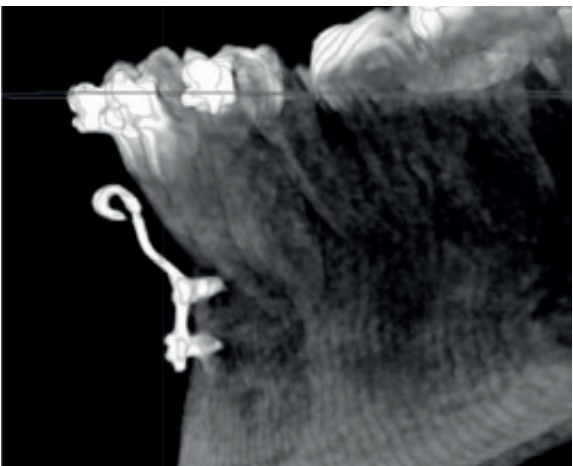
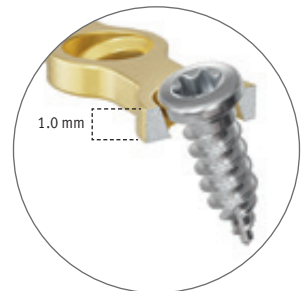
17.9 mm

0.6 mm

Illustrations are for illustrative purposes only and are not to scale, except where scale is indicated.

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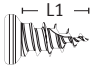
Continuously sharp thread flanks cut into the bone
Durchlaufend scharfe Gewindespitzen

Selfdrilling Screw tip
Selbstbohrende Schraubenspitze

ST P2

	L1	Ø 2.0 mm	
M 3:1	4 mm	ST-33-820-04	 L1 Ø
	5 mm	ST-33-820-05	
	7 mm	ST-33-820-07	
			M 3:1



BENEFIT® Screws | BENEFIT® Schrauben

BENEFIT® Orthodontic Screws, Titanium

BENEFIT® KFO-Schrauben, Titan

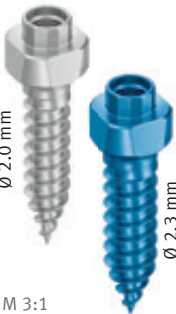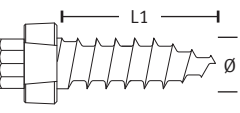ST P1

	L1	Ø 2.0 mm	Ø 2.3 mm	
 Ø 2.0 mm Ø 2.3 mm M 3:1	5 mm	ST-33-54205	-	 L1 Ø M 3:1
	7 mm	ST-33-54207	ST-33-54307	
	8 mm	ST-33-54208	-	
	9 mm	ST-33-54209	ST-33-54309	
	10 mm	ST-33-54210	-	
	11 mm	ST-33-54211	ST-33-54311	
	13 mm	ST-33-54213	ST-33-54313	
	15 mm	ST-33-54215	ST-33-54315	

BENEFIT® Orthodontic Screws, Titanium

BENEFIT® KFO-Schrauben, Titan

L	Ø 2.0 mm	
7 mm	ST-33-54207-2	P2 ST
9 mm	ST-33-54209-2	
11 mm	ST-33-54211-2	

BENEFIT® Screw Dispenser

BENEFIT® Schraubenspender

L	Ø 2.0 mm	
7 mm	ST-33-54207-10	P10 ST
9 mm	ST-33-54209-10	
11 mm	ST-33-54211-10	

Hybrid Hyrax® RPE with **BENEFit**® | Hybrid Hyrax® GNE mit **BENEFit**®

The **Hyrax**® Ring and **Hyrax**® Abutment facilitate the construction and fixation of a **Hybrid Hyrax**® appliance. It allows welding or soldering with the **Hyrax**® Screw as well as little rotation movements when connecting the appliance to the **BENEFit**® Screws. The **Hyrax**® Ring is supplied with 1 fixation screw each.

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BENEFit® Hyrax® Abutment **BENEFit**® Hyrax® Abutment

33-54463	
-----------------	--

bm^x – BENEFIT® Maxillary Xpander | bm^x – BENEFIT® Maxillary Xpander

After using the **BENEFIT®** Screw for the **Hybrid Hyrax®** appliance very successfully in the last years the **bm^x** now follows the Orthodontists request for an entirely **BENEFIT®** mini-implant anchored appliance for Rapid maxillary expansion on young patients. Using the De Franco Screwguide (Page 19) the distance and the parallelity of the **BENEFIT®** Screws can be exactly determined, directly on the patient. An impression is no longer necessary. Alternatively visualisation can of course be made by means of digital imaging or traditional impression taking.

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▶ Video

PSM –
BENEFIT® de Franco Drill Guide


BENEFIT® DIRECT Screws | BENEFIT® DIRECT Schrauben

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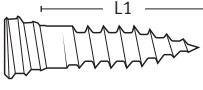
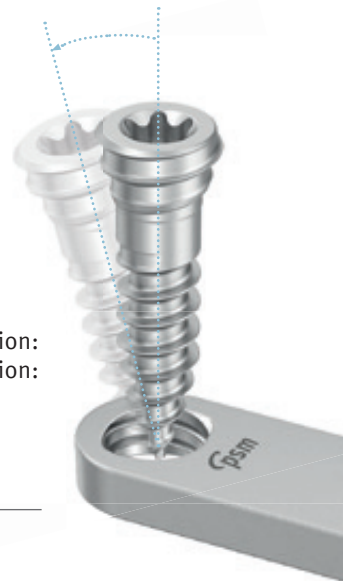
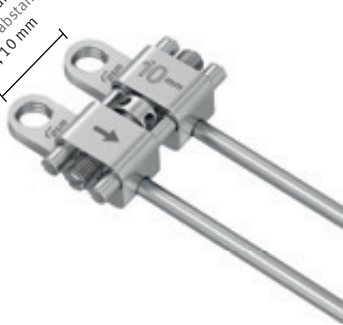
Plate distance:
Plattenabstand:
6, 8, 10 mm

▶ Video

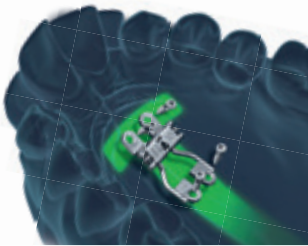PSM –
The smartlock
BENEFIT DIRECT
Expander

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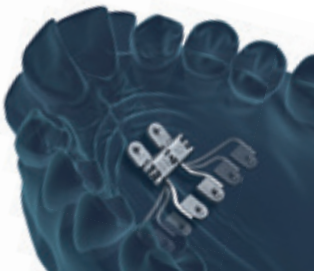
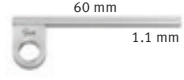
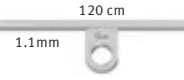
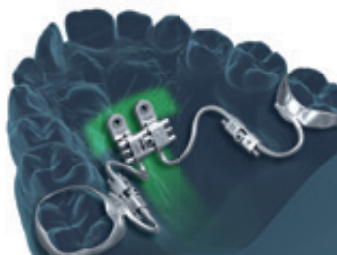
Sterile Packaging | Sterile Verpackung

Implants – in single sterile packaging

The outer and inner packaging contains labels indicating UDI code-like article number, LOT number, expiration date, etc. for the patient folder and for full traceability.

The sterile screws are contained in a small colour-coded screw carrier which protects the screw and at the same time acts as a pick up tray. This carrier is placed in a clear, sealed blister package. The screw can easily be removed from the carrier by means of the self retaining screwdriver.

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The instruments can be used for several **PSM Medical** products. In order to find the right instruments for your use, they are indicated in color.

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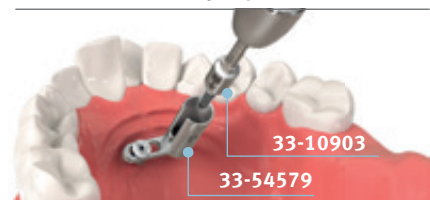
**BENEFIT® DIRECT TX Screwdriver
BENEFIT® DIRECT TX Schraubenhalter**

33-825-33	28 mm		
-----------	-------	--	--

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Laboratory Screwdriver, fixed
Labor Schraubendreher, fest

33-18288	■ ■	P1
----------	-----	----

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
Handles | Handgriffe

The manually turned unit is connected to your existing contra-angled handpiece (blue 1:1) that enables its use like an angled screwdriver.

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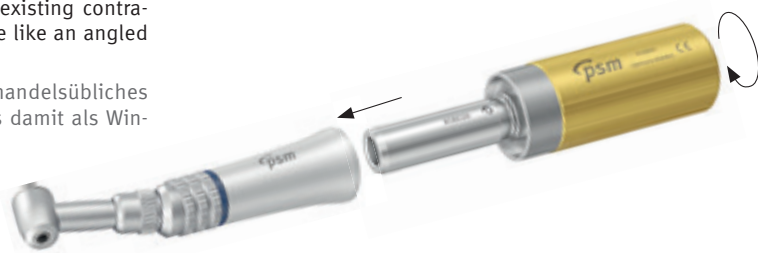Manually turned unit for contra-angled handpieces

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10-63025

Manually turned unit mod. to Pauls, with adjustable torque from 0 – 40 Ncm,

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33-54704

Multi purpose handle for the use with instruments with dental shaft end

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33-18236

Thumb screw for Dental mandrel with limited torque (10 Ncm)

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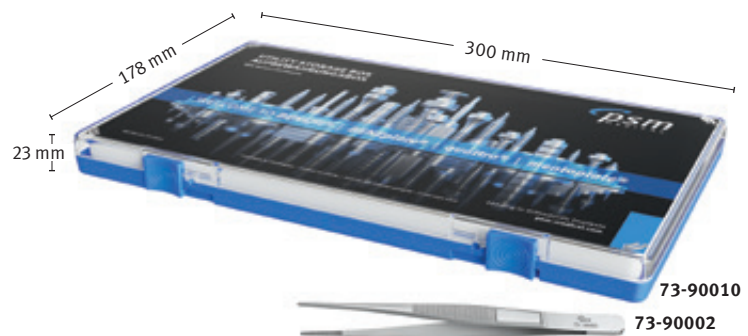
Catalogues | Kataloge

More innovative **PSM** products are presented in our other catalogues. Please request it from your local **PSM** distribution partner or download from our website.

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easy diver®

BENEfit® - System Handout

Specifications and design are subject to change without prior notice. All pictures, photos and product descriptions are the property of PSM Medical GmbH. Utilization and copies by third parties have to be authorized by PSM Medical GmbH. All rights reserved! All trademarks and brands mentioned in this brochure are subject to unrestricted regulations of the trade mark laws valid and right of ownership of the registered owner.

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BENEFIT®

No. 1 in skeletal anchorage.

**Precise Anchoring.
Maximum Flexibility.**

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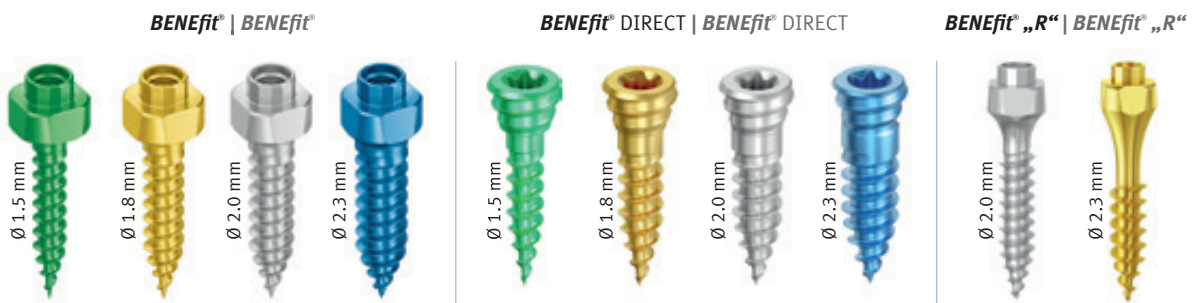
The **BENEFIT®** implant system from **PSM Medical** has been offering stable, cost-effective anchoring with a secure connection and flexible application options since 2007 – globally.

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BENEFIT® Screw Family **BENEFIT® Schrauben-Familie**

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PSM North America, Inc.