***smartlock***<sup>®</sup> feat. ***BENEFIT***<sup>®</sup> DIRECT

Leading in Orthodontic Implants  
[psm-medical.com](http://psm-medical.com)

WELCOME TO *BENEFIT*® CITY

*BENEFIT*®

C I T Y

## Table of Content | Inhaltsverzeichnis

English	Deutsch	Page   Seite
Introduction	Einführung	5
Difference <b>BENEFIT</b> ® & <b>BENEFIT</b> ® DIRECT	Unterschied <b>BENEFIT</b> ® & <b>BENEFIT</b> ® DIRECT	6
<b>BENEFIT</b> ® DIRECT Screws	<b>BENEFIT</b> ® DIRECT Schrauben	8
<b>MARPE</b> – Miniscrew Assisted Rapid Palatal Expansion	<b>MARPE</b> – Miniscrew Assisted Rapid Palatal Expansion	9
<b>MARPE</b> – Typodont Examples	<b>MARPE</b> – Modellbeispiele	10
Drills & Extension	Bohrer & Verlängerung	12
Screwholder & Handles	Schraubenhalter & Handgriffe	13
Battery Screwdriver iSD900	Akku Schraubendreher iSD900	14
Handle & Screwdriver Handle	Handgriff & Schraubendreher Handgriff	15
Micro Stix & Trays	Micro Stix & Behälter	16
<b>BENEFIT</b> ® Starter Set „M“	<b>BENEFIT</b> ® Starter Set „M“	17
Laboratory Products	Labor Produkte	17
<b>smartlock</b> ® – step by step	<b>smartlock</b> ® – Schritt für Schritt	19
Sterile Packaging & Catalogues	Sterile Verpackung & Kataloge	20

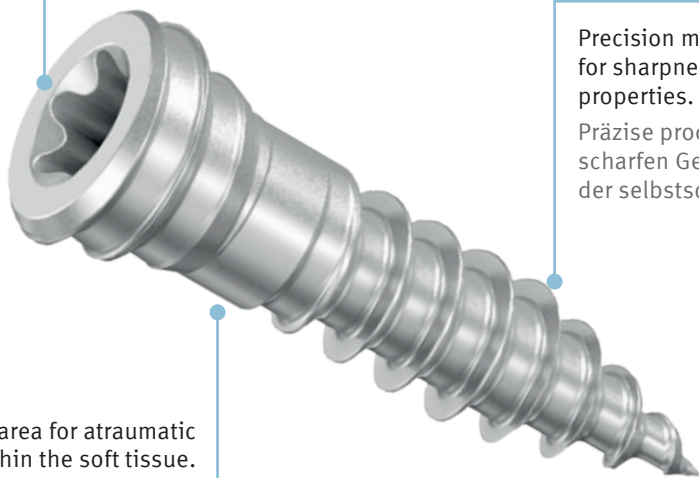
**BENEFIT® DIRECT Screws | BENEFIT® DIRECT Schrauben**

TX - self holding screw head with increased surface contact.

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Precision manufactured thread profile for sharpness and increased self tapping properties.

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Blocking thread for optimized compatibility with BENEFIT smartlock design

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## Difference **BENEFIT**® & **BENEFIT**® DIRECT

### Unterschied **BENEFIT**® & **BENEFIT**® DIRECT

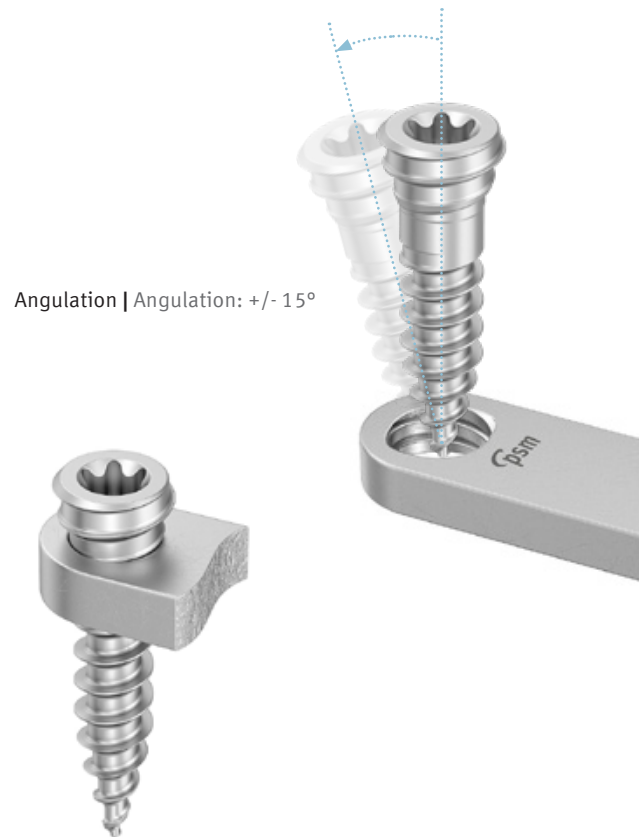
**BENEFIT**

#### IMPLANT FIRST

The traditional workflow to insert the classic **BENEFIT**® Implants first and then build the appliance on top has its proven advantages when it comes to distalization (Beneslider) and mesialization (Mesialslider) as there are no points to fix the appliance in the palate before screw insertion.

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
**BENEFIT** DIRECT

#### **BENEFIT**® DIRECT

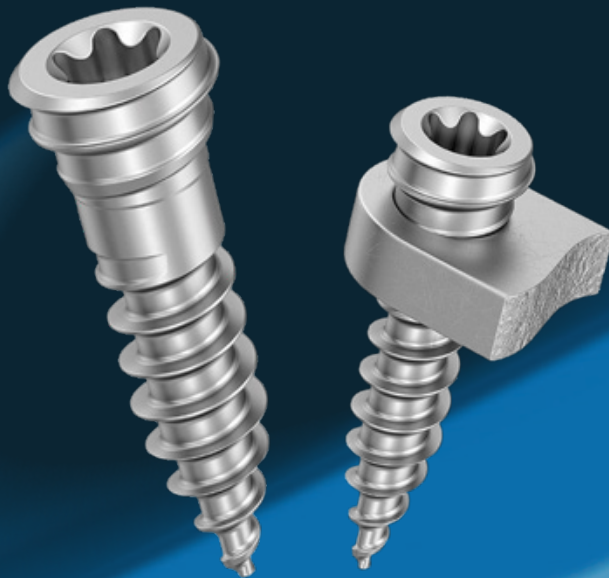The new **BENEFIT**® DIRECT Screws show their big advantages when an **APPLIANCE FIRST workflow** is preferred. Without any additional software the appliance (i.e. Hybrid Hyrax, MARPE) can be prepared on the patient model with our laboratory tools and pre-welded **bmX** DIRECT Expanders. The appliance can be fixed on the molars and the **BENEFIT**® DIRECT Screws secure the appliance safely in the position desired. The threaded head design ensures a secure and firm connection between the DIRECT plates and the implant without any tilting.

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**Products**  
**Produkte**

**BENEFIT® DIRECT Screws | BENEFIT® DIRECT Schrauben**

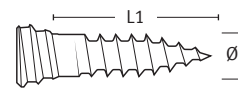
BENEFIT® DIRECT Screws, Titanium

BENEFIT® DIRECT Schrauben, Titan

ST P1

L1	Ø 1.8 mm	Ø 2.0 mm	Ø 2.3 mm
7 mm	ST-33-64187	ST-33-64207	ST-33-64307
9 mm	ST-33-64189	ST-33-64209	ST-33-64309
11 mm	ST-33-64191	ST-33-64211	ST-33-64311
13 mm	-	ST-33-64213	ST-33-64313
15 mm	-	ST-33-64215	ST-33-64315

M 3:1

BENEFIT® DIRECT Screws, Titanium

BENEFIT® DIRECT Schrauben, Titan

L	Ø 2.0 mm
7 mm	ST-33-64207-2
9 mm	ST-33-64209-2
11 mm	ST-33-64211-2

P2

ST

Direct screw removal from the sterile holder.

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

## MARPE – Miniscrew Assisted Rapid Palatal Expansion MARPE – Miniscrew Assisted Rapid Palatal Expansion

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

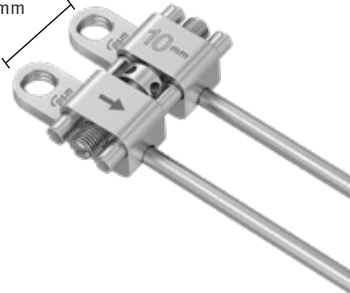
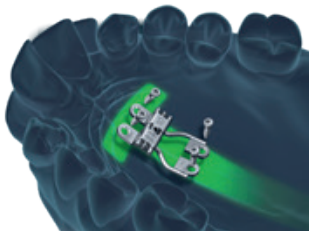
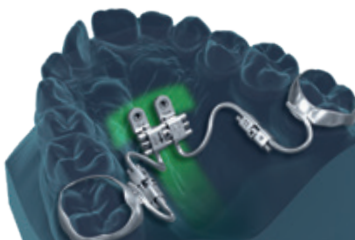
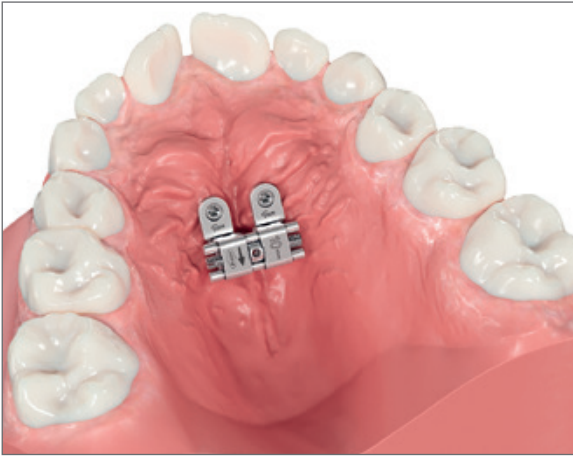
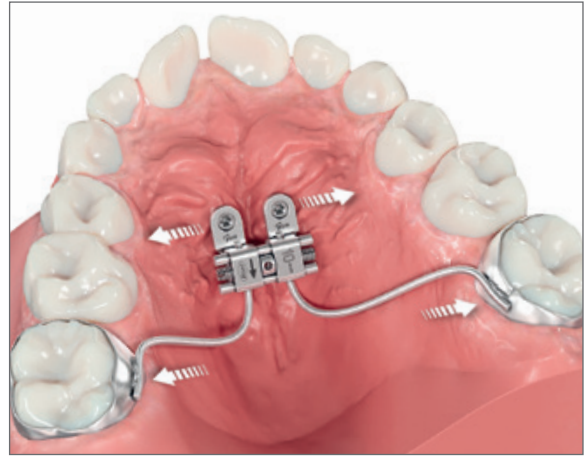
**bm<sup>x</sup>** DIRECT Quadexpander | **bm<sup>x</sup>** DIRECT Quadexpander

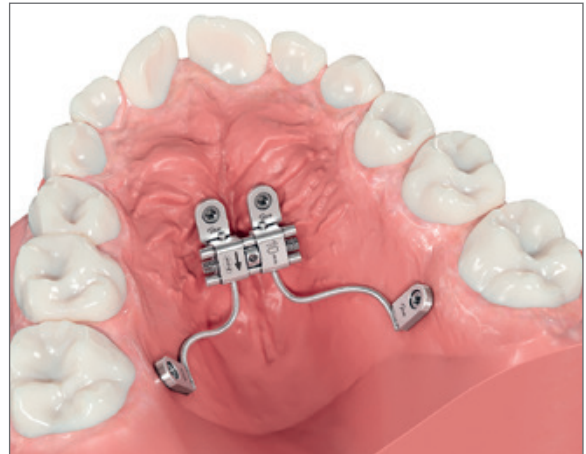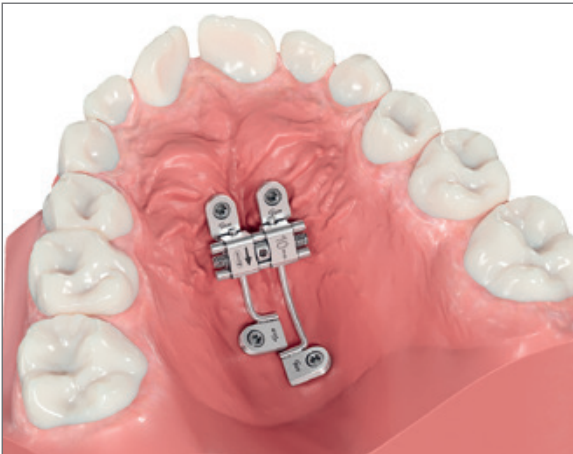**Hybrid Hyrax®** Distalizer | **Hybrid Hyrax®** Distalizer

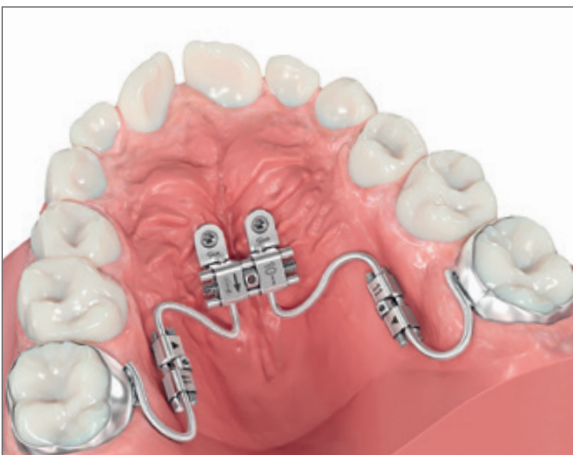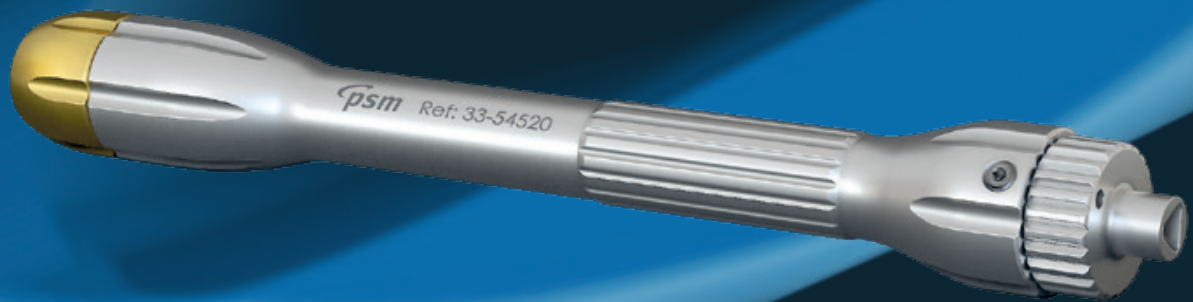**Instruments**  
**Instrumente**

## Drills | Bohrer

The instruments can be used for several **PSM Medical** products. In order to find the right instruments for your use, they are indicated in color.

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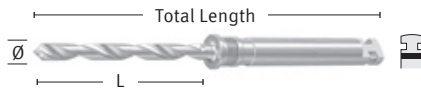
**Bohrer verlängerung**, + 16 mm  
(max. 600 U/min)

33-18244  

The extension of 16 mm works for all rotary instruments of the quattro® and **BENEFIT®** System with a latch type DENTAL attachment (max. rpm 600).

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**TX 2.0 Screwholder** for 10-61000  
**TX 2.0 Schraubenhalter** für 10-61000

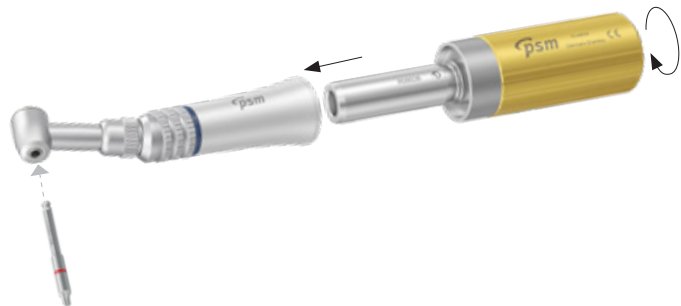
**Screwdriver handle**  
**Schraubendreher Handgriff**


The manually turned unit is connected to your existing contra-angled handpiece (blue 1:1) that enables its use like an angled screwdriver.

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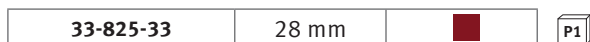## Battery Screwdriver – iSD900 | Akku Schraubendreher iSD900

### Cordless Battery Screwdriver with Torque Calibration System for the controlled fixation of PSM Miniscrews.

The iSD 900 will guarantee precise torque control due to the unique – easy to use – torque calibration system (10 – 40 Ncm). Time-saving and precise, riskfree insertion and tightening of screws and abutments. The complete set contains a reliable and user friendly handpiece for Dental attachments. Reverse mode is indicated with sound. Standard rechargeable AAA Ni-Mh batteries can be used.

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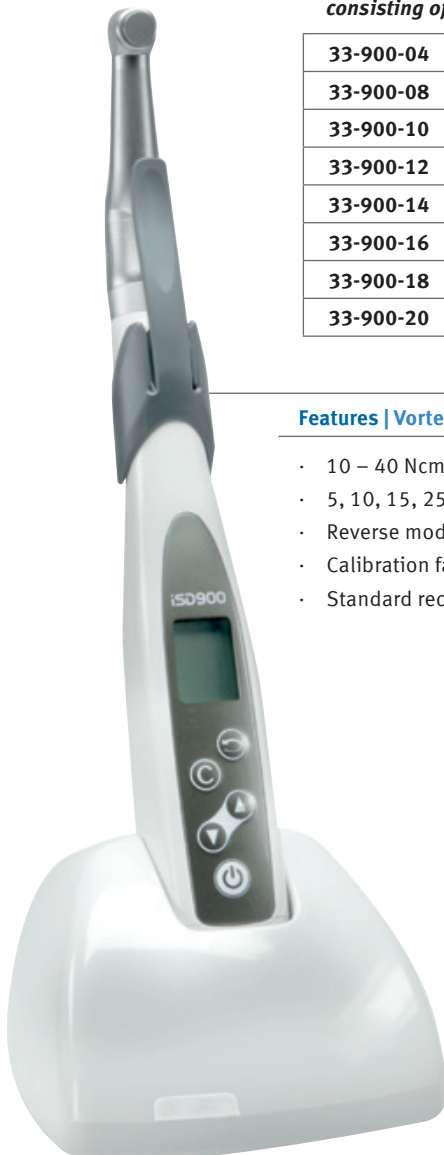
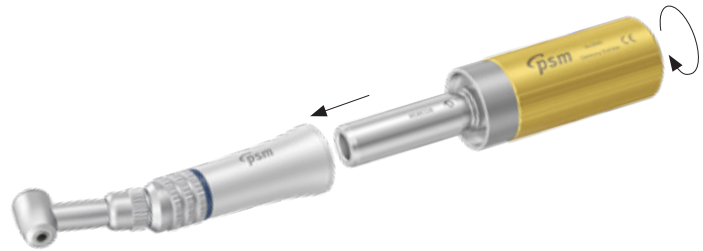
10-63025

**Manually turned unit mod. to Pauls**, with adjustable torque from 0 – 40 Ncm

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33-54704

**Multi purpose handle** for the use with instruments with dental shaft end

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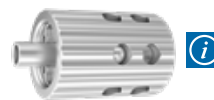
33-18236

**Thumb screw** for Dental mandrel with limited torque (10 Ncm)

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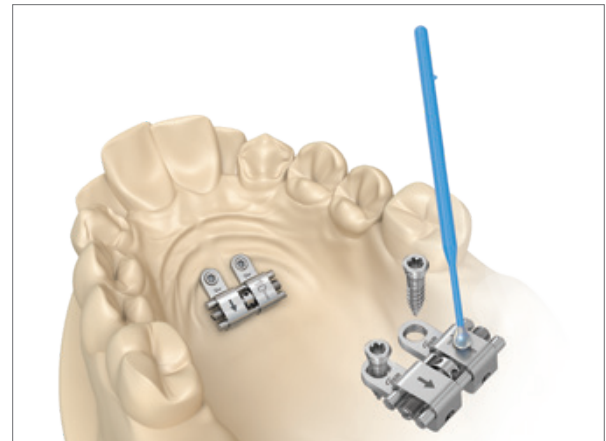
33-64890

P64

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**BENEFIT® Starter Set Mini-Tray**,  
black, empty

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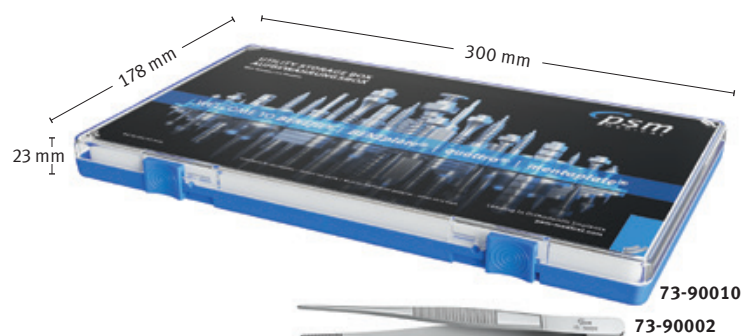
73-31990

**BENEFIT® Utility Storage Tray Set**,  
consisting of 73-90010 and 73-90002

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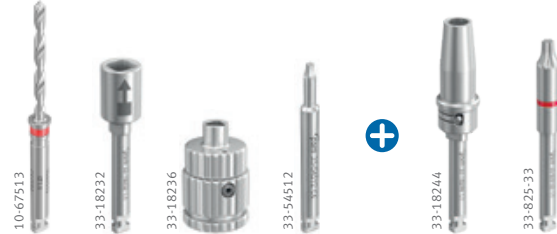
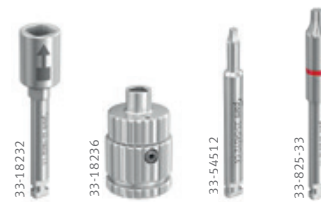
**Activating Screwdriver\*** for mobilizer,  
not autoclavable, 13 cm

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## Laboratory Products | Labor Produkte

**smartlock®** Lab Cone Pin,  
**smartlock®** Labor Kegel-Pin

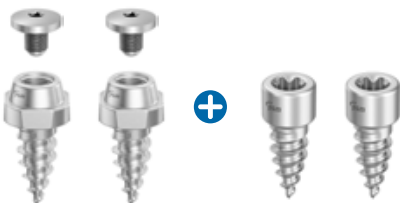
33-64880	P2
----------	----

**smartlock®** Laboratory Inside-Pin,  
**smartlock®** Labor Innen-Pin

33-64881	P2
----------	----

**smartlock®** - Set, Lab & Cone Pin  
**smartlock®** Set, Kegel- und Innen-Pin

33-64879	P2
----------	----


Video

**smartlock BENEfit®**  
DIRECT Expander

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
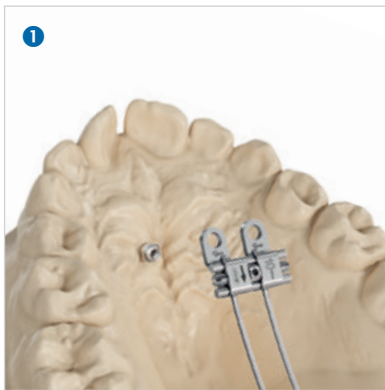
**Laboratory | Labor**

- 1 Place the cone pin in the laboratory model
- 2 Fix the device and insert the Inside-Pin
- 3 Adaptation of the appliance to the anatomical conditions of the patient
- 4 Create a thermoforming foil and cut out the front screw surface

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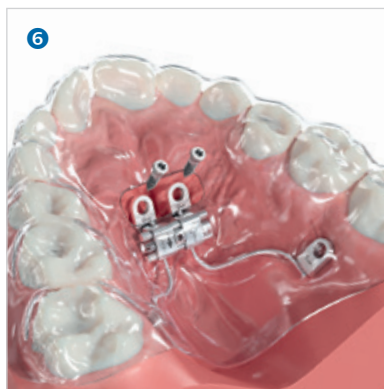
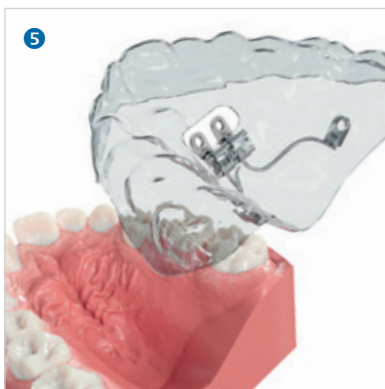
**Patient | Patient**

- 5 Thermoforming foil with appliance is placed on the patient
- 6 Anterior Smartlock **BENEFIT®** DIRECT screws are inserted
- 7 Thermoforming foil can be removed and the posterior screws can be set
- 8 Start of expansion

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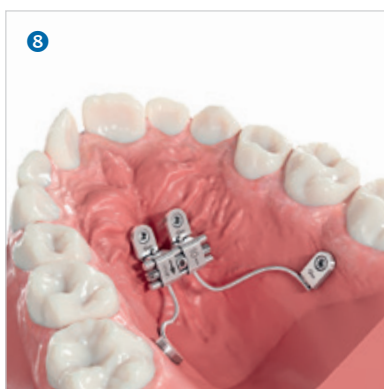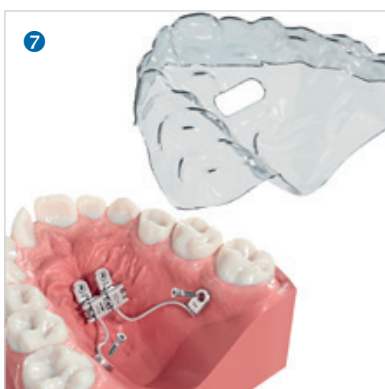smartlock **BENEFIT®**  
DIRECT Screws

## Sterile Packaging | Sterile Verpackungen

### Implants – in single sterile packaging

The outer and inner packaging contains labels indicating UDI code-like article number, LOT number, expiration date, etc. for the patient folder and for full traceability. The sterile screws are contained in a small colour-coded screw carrier which protects the screw and at the same time acts as a pick up tray. This carrier is placed in a clear, sealed blister package. The screw can be removed easily from the carrier by means of the self retaining screwdriver.

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## Catalogue | Kataloge

More innovative **PSM** products are presented in our other catalogues. Please request it from your local **PSM** distribution partner or download from our website. [www.psm-medical.com](http://www.psm-medical.com).

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## BENEFIT

With the introduction of the **BENEFIT**® System **PSM Medical** has considerably enhanced the therapeutic diversity of the treatments available for Mesialization, Distalization, RPE as well as uprighting or anchorage of the dentition. With **BENEFIT**®, **PSM Medical** was the first company to offer a full range of skeletal anchorage appliances for the palate – a trend that has become „state of the art“ in Orthodontics.

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## quattro

The **quattro**® Mini-Anchor System is known as one of the first and most successful orthodontic Mini-Implant Systems worldwide. Based on this system the **BENEFIT**® screws were developed. This enables the user to utilize the entire scope of orthodontic implantation with unmatched possibilities and compatible instrumentation.

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## mentoplate

The **mentoplate**® system provides a perfect solution for enhanced stability with special orthodontic bone anchored plates for class III correction. The System includes Titanium plates which are fixed to the bony surface with selfdrilling screws.

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THE GUIDING SYSTEM FOR TABS

### Digital skeletal anchorage | Digitale skelettale Verankerung

A new and innovative method for the secure insertion of the **BENEFIT**® mini screws. Implantation and appliance placement in one session.

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## BENEFIT

For more information please refer to our **BENEFIT**® System – Handout or our website [www.psm-medical.com](http://www.psm-medical.com).

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KAT-012-ED

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[www.psm-medical.com](http://www.psm-medical.com)

Specifications and design are subject to change without prior notice. All pictures, photos and product descriptions are the property of PSM Medical GmbH. Utilization and copies by third parties have to be authorized by PSM Medical GmbH. All rights reserved! All trademarks and brands mentioned in this brochure are subject to unrestricted regulations of the trade mark laws valid and right of ownership of the registered owner.

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**PSM North America, Inc.**